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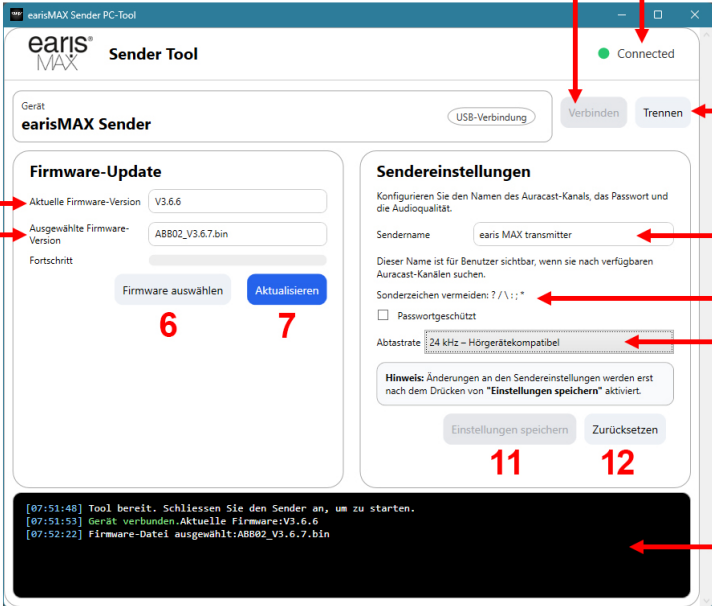
The screenshot shows the 'earisMAX Sender PC-Tool' window. It has a title bar with standard Windows window controls. The main area is divided into two panels: 'Firmware-Update' on the left and 'Sendereinstellungen' on the right. At the top right, there's a 'Connected' status indicator. Below it are 'Verbinden' and 'Trennen' buttons. The 'Firmware-Update' panel shows 'Aktuelle Firmware-Version' as V3.6.6 and 'Ausgewählte Firmware-Version' as ABB02\_V3.6.7.bin. There are buttons for 'Firmware auswählen' and 'Aktualisieren'. The 'Sendereinstellungen' panel has fields for 'Sendername' (earis MAX transmitter), a checkbox for 'Passwort geschützt', and a dropdown for 'Abtastrate' (24 kHz - Hörgerätekompatibel). At the bottom of this panel are 'Einstellungen speichern' and 'Zurücksetzen' buttons. A log window at the bottom shows system messages. Red arrows with numbers 1-13 point to specific UI elements: 1 points to 'Verbinden', 2 to 'Connected', 3 to 'Trennen', 4 to 'Aktuelle Firmware-Version', 5 to 'Ausgewählte Firmware-Version', 6 to 'Aktualisieren', 7 to 'Firmware auswählen', 8 to 'Sendername', 9 to 'Passwort geschützt', 10 to 'Abtastrate', 11 to 'Einstellungen speichern', 12 to 'Zurücksetzen', and 13 to the log window.

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# Setting up and using the earisMAX Auracast transmitter PC tool

EN

Download the zip-packed PC tool (Windows only) from the Humantechnik website:  
<https://www.humantechnik.com/downloads/earisMAX-PC-tool.zip>

Before running the tool, the earisMAX Auracast transmitter must be connected to your computer using a USB-C cable capable of data transfer. Extract the folder to a directory of your choice. The programme “earisMAX\_Setup.exe” is now located in this directory. Launch the programme by double-clicking it. Once you have selected the installation language, the tool will be installed.

A shortcut will be created on the desktop; you can launch the tool by double-clicking this shortcut:

The screenshot shows the 'earisMAX Sender Tool' window. It has a title bar with standard Windows window controls. Below the title bar, there's a status bar showing 'Connected' with a green dot. The main area is divided into two panels. The left panel is titled 'Firmware-Update' and contains fields for 'Aktuelle Firmware-Version' (V3.6.6) and 'Ausgewählte Firmware-Version' (ABB02\_V3.6.7.bin). Below these are buttons for 'Firmware auswählen' (6) and 'Aktualisieren' (7). The right panel is titled 'Sendereinstellungen' and contains fields for 'Sendername' (earis MAX transmitter) (8), a password field (9), and a sample rate dropdown (10) set to '24 kHz - Hörgerätekompatibel'. Below these are buttons for 'Einstellungen speichern' (11) and 'Zurücksetzen' (12). At the bottom, there's a log window (13) showing system messages. Red arrows point from numbers 1 to 13 to specific UI elements: 1 points to the 'Verbinden' button, 2 points to the 'Connected' status, 3 points to the 'Trennen' button, 4 points to the 'Aktuelle Firmware-Version' field, 5 points to the 'Ausgewählte Firmware-Version' field, 6 points to the 'Firmware auswählen' button, 7 points to the 'Aktualisieren' button, 8 points to the 'Sendername' field, 9 points to the password field, 10 points to the sample rate dropdown, 11 points to the 'Einstellungen speichern' button, 12 points to the 'Zurücksetzen' button, and 13 points to the log window.

- 1 - Connect
- 2 - Connection status
- 3 - Disconnect
- 4 - Firmware version on device
- 5 - Selected firmware version
- 6 - Locate new firmware
- 7 - Update
- 8 - Transmitter name (name of Auracast-stream)
- 9 - Set Auracast-stream-password
- 10 - Select samplerate
- 11 - Save settings
- 12 - Reset
- 13 - Statusreport

Click on ‘Connect’ (1) to connect the device to the app. The connection status is shown at (2). Once the connection is successful (‘Connected’), the currently installed firmware version is displayed at (4).

## Firmware update

If a more recent firmware version is available (which can be viewed on the product website at <https://www.humantechnik.com/produkte/tv-hoersysteme/earismax-das-system/earismax-bluetooth-sender>), the ZIP-compressed firmware package can be downloaded from there.

Unzip the folder into a directory of your choice. This directory will now contain a file named ‘ABB02\_version number.bin’. Click ‘Select firmware’ (6) to navigate to and select the directory containing the new firmware.

The selected firmware version should now be displayed under ‘Selected firmware version’ (5). This is transferred to the earisMAX Auracast transmitter by clicking ‘Update firmware’ (7). Do not switch off the computer or disconnect the transmitter’s USB cable during this process – otherwise the device may become unusable.

The process may take up to 10 minutes. Once complete, a confirmation message will be displayed in the log window (13).

## Encryption of the Auracast stream and setting the sample rate

Access to the Auracast stream generated by the transmitter can be restricted by setting a password. The name of the Auracast stream can be set under (8) and the password under (9). If the ‘Password-protected’ box is ticked, a password can be set. Please use only letters and numbers for this. The password should consist of 4 to 8 characters. If you want the stream to be freely accessible to all listeners within range, untick the ‘Password-protected’ box.

The sample rate can be selected under (10). If any of the Auracast receivers are hearing aids, a setting of 24K may be necessary. The default setting of 48K is designed to ensure high-quality audio transmission.

These settings are transferred to the transmitter by clicking ‘Save settings’ (11). The changes can be detected via the earisMAX app (or other Auracast relay apps) when receiving the stream.

By clicking ‘Reset’ (12) resets the stream name, password and sample rate to their default values (default stream name, no password, 48K).

Important: Once all operations are complete, please disconnect the transmitter from the programme by clicking ‘Disconnect’ (3).

The programme can then be closed and the transmitter removed from the computer.

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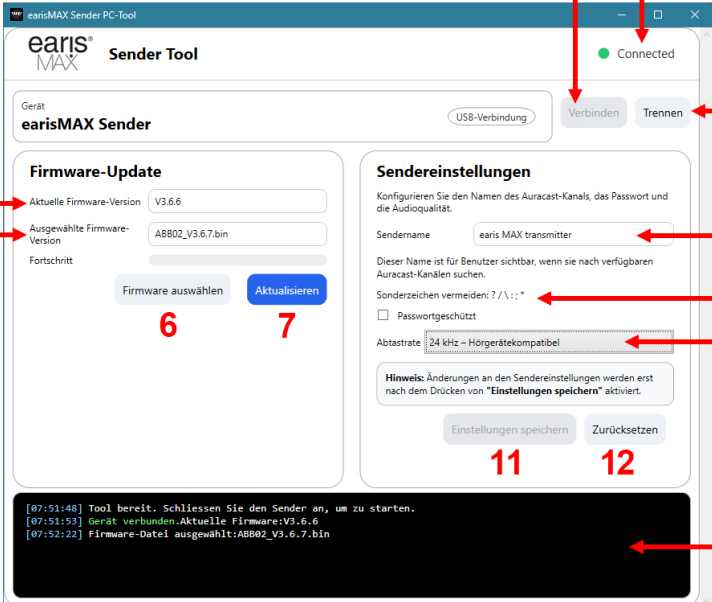
The screenshot shows the 'earisMAX Sender Tool' window. It has a top bar with the 'earisMAX' logo and 'Sender Tool' text. Below this, there's a 'Gerät' (Device) section showing 'earisMAX Sender' and a 'USB-Verbindung' (USB connection) status. A green dot and the word 'Connected' are visible. To the right of this are buttons 'Verbinden' (Connect) and 'Trennen' (Disconnect). Below the device section are two main panels: 'Firmware-Update' and 'Sendereinstellungen' (Sender settings). The 'Firmware-Update' panel shows 'Aktuelle Firmware-Version' (Current firmware version) as 'V3.6.6' and 'Ausgewählte Firmware-Version' (Selected firmware version) as 'ABB02\_V3.6.7.bin'. There are buttons 'Firmware auswählen' (Select firmware) and 'Aktualisieren' (Update). The 'Sendereinstellungen' panel has fields for 'Sendername' (Sender name) set to 'earis MAX transmitter', a password field, and a dropdown for 'Abtastrate' (Sampling rate) set to '24 kHz - Hörgerätekompatibel'. There are buttons 'Einstellungen speichern' (Save settings) and 'Zurücksetzen' (Reset). At the bottom, there's a black log window showing system messages. Red arrows with numbers 1 through 13 point to specific elements: 1 points to 'Verbinden', 2 to 'Connected', 3 to 'Trennen', 4 to 'Aktuelle Firmware-Version', 5 to 'Ausgewählte Firmware-Version', 6 to 'Firmware auswählen', 7 to 'Aktualisieren', 8 to 'Sendername', 9 to the password field, 10 to 'Abtastrate', 11 to 'Einstellungen speichern', 12 to 'Zurücksetzen', and 13 to the log window.

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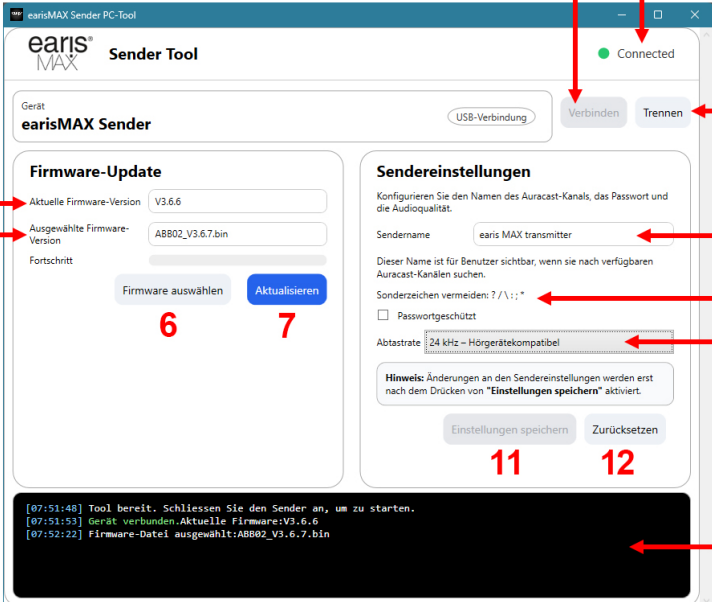
The screenshot shows the 'earisMAX Sender Tool' window. It has a top bar with 'Connected' status. Below are two main panels: 'Firmware-Update' on the left and 'Sendereinstellungen' on the right. The 'Firmware-Update' panel shows 'Aktuelle Firmware-Version' as V3.6.6 and 'Ausgewählte Firmware-Version' as ABB02\_V3.6.7.bin. The 'Sendereinstellungen' panel shows 'Sendername' as 'earis MAX transmitter', a password field, and a sample rate of 24 kHz. A log window at the bottom shows connection and firmware update status.

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