TAYO

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1 General instructions

1.1 Introduction

These operating instructions contain information about how to use the shower stool/shower chair safely and properly.

Read the operating instructions thoroughly prior to the use of the product and follow all instructions.

Keep all documents in a safe place.

1.2 Proper use

The shower stool/chair is used solely and exclusively as an aid to shower adolescents and adults in the bathroom. It allows persons with reduced mobility to shower sitting down. There are models available with backrest and armrests for persons with a more reduced mobility. Any other form of use is not permitted.

The shower stool/chair may not be used by children. The shower stool/chair is not suitable for persons who are smaller than 146 cm or weigh less than 40 kg (BMI < 17).

Locations in which the shower stool/shower chair is intended to be used include retirement and nursing homes, hospitals, rehabilitation centres and private households.

The **indication** is indicated for: inability to stand / unsteady standing / limb loss/defect/deformation.

Contraindication is indicated for: false sensations / severe disequilibrium / inability to sit.

1.3 Disposal

The product consists of recyclable materials.

Old appliances and packaging must be disposed of and recycled in accordance with the valid national regulations.

1.4 CE mark

The technical requirements for the product are set out in ISO 17966 (personal hygiene aids) and are met. The product complies with the medical products directive (EU) 2017/745.

1.5 Reuse

The product is suitable for reuse. The amount of reuse depends on the condition of the product in question.

Prior to reuse, follow the manufacturer's reconditioning and hygiene guidelines, and check the technical and functional safety of the product as well. In Germany, the DIETZ GROUP offers a professional hygiene service for reconditioning the devices in accordance with the MDR. Further information can be requested from kundenservice@dietz-group.de.

1.6 Service life

The product is designed for a service life of at least 3 years, provided that it is used in accordance with its intended use and that all cleaning and safety instructions are observed. Further use or inspection/repair of the product after this period is conducted at the discretion and responsibility of the operator.

2 Safety instructions

Do not overload the shower stool/chair (max. user weight: 130 kg).

Position shower stool/chair only on level surface.

Do not use shower stool/chair as a climbing assistance nor stand on it.

Do not load the shower stool/chair one-sidedly.

All chair legs must be adjusted to the same height.

Check shower stool/chair for damage prior to every use. Do not use a damaged shower stool/chair.

In case of severely reduced mobility, use the shower stool/chair with the help of an assistant.

Hot water, sunlight or hot air can cause the surface of the product to become hot. Risk of burns!

Serious incidents relating to the product must be reported to the manufacturer and the competent authority.

3 Description

The product is available in the following versions:

- Shower stool (Fig. 1A)
- Shower stool with armrests (Fig. 1B)
- Shower chair with backrest (Fig. 1C)
- Shower chair with backrest and armrests (Fig. 1D)
- B-protect (with antimicrobial coating) (without Fig.)

The shower stool consists of a seat section and four chair legs. The seat section has a hygiene cutout that makes washing the private parts easier. The ends of the chair legs are provided with large rubber pads that enhance anti-slip safety. A washable seat cushion (1, Fig. 5) is optionally available.

Depending on the model, a backrest and/or armrests will provide additional support. The backrest can be attached at the front or at the rear of the seat section (depending on the desired orientation of the hygiene cutout).

3.1 Technical data

Refer to "Technical data" at the end of the operating instructions.

3.2 Label

The label (Fig. 6) is attached to the bottom side of the seat section and contains the following information (exemplary illustration):

1 manufacturer; **2** max. user weight; **3** follow the operating instructions; **4** indoor use only; **5** date of manufacture (YYYY-MM-DD); **6** mark of conformity; **7** medical device; **8** UDI = Unique Device Identifier: (01) UDI-DI/GTIN, (10) batch / lot number, (11) date of manufacture (YYMMDD); **9** LOT = batch / lot number; **10** REF = item number; **11** TYP = model

4 Use

CAUTION Do not allow any oily or strongly soapy substances to get to the rubber pads. Otherwise, the shower stool/chair may slip.

CAUTION When using the shower stool/chair, keep a still position and avoid excessive body movements. Otherwise, there is a risk of slipping.

1. Place the shower stool/chair in the shower.
2. Sit down on the dry seat section.

5 Assembly

5.1 Attachment of chair legs

CAUTION Risk of crushing while fitting the chair legs. Fit the chair legs carefully.

1. Pull clip (2, Fig. 2) out of the chair leg.
2. Push chair leg (3, Fig. 2) as far as it will go onto the connecting piece (1, Fig. 2) of the seat section. When doing this, the broad side of the rubber pad (6, Fig. 2) must point inwards.
3. Push clip (2, Fig. 2) through the hole in the chair leg and the connecting piece until it snaps in.
4. Pull clip (4, Fig. 2) out of the chair leg and the extendable section.
5. Pull extendable section (5, Fig. 2) out to the desired seat height.
6. Push clip (4, Fig. 2) through the hole in the chair leg and the extendable section until it snaps in.

5.2 Attachment of armrests

Press armrest (1, Fig. 3) into recess (3, Fig. 3) of the seat section until the locking pieces (2, Fig. 3) snap in.

5.3 Attachment of backrest

Press backrest (1, Fig. 4) into front or rear recesses (3, Fig. 4) of the seat section until the locking pieces (2, Fig. 4) snap in.

6 Disassembly

Armrests/backrest: Turn over the seat section, press locking pieces inwards and pull armrest/backrest out.

Chair legs: Pull clip out and remove chair leg.

7 Maintenance and care

7.1 Care

NOTICE Do not use any scouring agent. Otherwise, the plastic surfaces may be damaged.

The product can be cleaned with lukewarm water mixed with a mild detergent (without solvents and of a pH value of 5-9). Care for the plastic pieces with an off-the-shelf plastic cleaner.

If required, the shower stool/chair can be disassembled and the (optional) seat cushion removed to clean/disinfect the parts individually. The seat cushion can be washed at 40 °C in the washing machine but may not be dried in a tumble dryer.

If necessary, as well as if using the product for more than one person, the chair surface can be disinfected with an off-the-shelf disinfectant (max. 70 % alcohol) **after cleaning**. Depending on the disinfectant used, the disinfectant will be sprayed on or applied with a cloth. Follow the disinfectant or detergent manufacturer's instructions.

When adhering to the specified care and safety instructions, the product is maintenance-free.

7.2 Removal of seat cushion

Release all plugs (2, Fig. 5) of the seat cushion (1, Fig. 5) from the seat section (3, Fig. 5).

7.3 Attachment of seat cushion

1. Place and align seat cushion (1, Fig. 5) on seat section (3, Fig. 5).
2. Press all plugs (2, Fig. 5) into the holes in the seat section.

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© DIETZ GmbH, Karlsbad, Deutschland
DIETZ GmbH, Karlsbad, Deutschland.
© DIETZ GmbH, Karlsbad, Germany
DIETZ GmbH, Karlsbad, Germany.
© DIETZ GmbH, Karlsbad, Alemania
DIETZ GmbH,– Karlsbad, Alemania.