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Rollator Standard

REF 11442 020

rot

PZN 03799819

HMV-Nr. 10.50.04.1115

REF 11442 035

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REF 11442 035

silver, with cane holder

Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place!
If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

Intended Use

The rollator supports the mobility of people with restricted mobility, allowing them a greater radius of action. Movement has a positive effect on circulation, coordination, balance and well-being. The rollator is suitable for indoor and outdoor use.

Indication

A rollator is suitable for people with restricted mobility (e.g. walking difficulties, severely restricted walking endurance) who still have sufficient walking ability, strength and stability to walk independently with the support of a rollator.

Contraindication

The rollator should not be used by persons with sensory impairments, severe balance problems or an inability to sit. The user must have sufficient cognitive and physical abilities to be able to operate a rollator.

⚠ Safety Indications ⚠

- Talk to your doctor or specialist retailer to see whether this Rollator is optimally suitable for you and your needs.
- Check that the Rollator is in perfect condition every time before you use it. Check the following components and make sure:
 - all screws, nuts and other parts are firmly tightened
 - all wheel surfaces are intact
 - the height adjustment screws are firmly tightened
 - the brakes function both as parking brakes and hand brakes
 - there are no visible signs of wear on the frames, welded joints and plastic elements
 - wheels and wheel forks should be free of dirt and turn smoothly.
 Do not use a damaged or defective Rollator!
- Ensure that the Rollator is correctly engaged before every use.
- Always activate the parking brake when you stop or park the Rollator. Always use the hand brakes slowly, evenly and both together.
- The braking effect can be reduced if the tyre surface is worn. Inspect the braking action of the brakes regularly.
- The Rollator should only be used on firm and even surfaces. There is a risk of slipping on dirty, wet, frosty or snow-covered ground! The braking effect is also limited. Please adapt the way you move and walk to these dangers.
- On smooth surfaces such as tiles, stone flooring, PVC etc. the wheels can block when braking under certain circumstances. Therefore please check the braking performance of your Rollator carefully on these floors and adjust the way you move and walk accordingly. If you wish to sit on the Rollator you should also make sure that the wheels have a sufficient hold on the surface when the brakes are applied and that the Rollator cannot slide away.
- The Rollator should not be used as a support when getting up from a crouching or sitting position!
- The maximum permissible load of the rollator is 130 kg. This includes the max. luggage load of 10 kg in the basket.
- The basket included in the scope of delivery is designed for a max. luggage load of 10 kg. The basket must not be overloaded. Only use the basket for transporting objects.
- The rollator must not be used outside its intended use or for transporting objects or persons. A maximum of 10 kg may be transported in the basket provided for this purpose.
- Ensure that the hand grips are set to the correct height for the user in order to avoid incorrect loads or falls. Always hold both handles, otherwise you could lose your balance. Walk between the hand grips. When walking, be careful not to bump your feet against the wheels or step assistance.
- Only use the rollator at normal walking speed, walk between the handles and apply an even load to the rollator. Failure to do so can cause the front wheels to wobble and increase the risk of falling.
- Make sure that your hands or clothes do not come into contact with the moving parts of the Rollator! Be particularly careful when folding and unfolding the Rollator. This applies in particular to hinges under the seat or in the middle of the cross brace.
- The rollator should be used on the pavement. The rollator must not be used in moving traffic. For your personal safety in road traffic, it is recommended to wear additional reflectors on your clothing in addition to the reflectors already attached to all four sides of the rollator.
- Do not sit on the rollator when travelling on public transport. Park the rollator with the brakes on so that it cannot fall over or roll away. Look for a seat close by.
- If you have to cross (tram) tracks, drive over them at a right angle and as quickly as possible. at a right angle and cross them quickly. Make sure that the wheels do not get caught in the rails.
- Be careful near children. Children should not play with the Rollator!
- Do not hang any handbags or carrier bags on the handles, the balance of the Rollator can be impaired if you do.
- Do not carry out any adjustments or modifications, which impair the safety of the design. Parts such as wheels, handles or brake cables may only be replaced by a specialist retailer.
- Only use accessories approved by the manufacturer. Please also read the instructions for use for each accessory.
- Do not carry or pull the Rollator by the brake cables.
- In strong sunlight, parts of the rollator, e.g. handles or seat, can heat up. In these cases, skin contact should be avoided to prevent burns. Make sure to cover your rollator or park it in a shady place.
- Please prevent skin contact with the Rollator if the Rollator has been in cold temperature for a long period of time in order to prevent freezing.
- Keep the rollator away from naked flames.

- Please prevent running into door frames, furniture, etc. with the Rollator and check the secure position of the Rollator when parked. The frame, connection elements and welding seams may be damaged if the Rollator tips over.
- Please also prevent placing stress on the Rollator when folded together! No objects may be placed on the folded rollator during transport, e.g. in

the car. This could damage the frame, connection elements and welding seams.

- Do not use the Rollator in stairs or escalators.
- Do not stand on the rollator.
- Do not lean on the brake lever.
- If you have any queries, please refer to your dealer.

Remove all packaging material. Check the product has not been damaged in transit before using it for the first time. Pay particular attention to ensuring that the wheels and wheel suspension, screw and plastic connections and brake cables are intact. The rollator is delivered pre-assembled. Before using for the first time, check that the brakes work properly; they may need to be readjusted. The hand grips must be adjusted to the appropriate height for the user. Please dispose of the packaging material correctly.

pic. 1


Contents/Description of the parts:

1. Brake cable
2. Tray
3. Seat
4. Fold lock with spring
5. Shopping basket
6. Baggage compartment
7. Handle and brake
8. Coupling clamp
9. Security lever
10. Brake cable adjusting screw
11. Crossbar
12. Reflectors
13. Back wheel, rigid
14. Step support
15. Front wheel, steerable

Assembly

Assemble the rollator as followed:

1. Fold the frame as described under »Unfold/Fold«.
2. Position the rollator so that the steerable front wheels point to the front.
3. Push the handlebars at least 9 cm into the tubing of the walking frame (up to the ridged mark of the handlebar).
4. The screw on the locking lever is fitted from inside towards the outside, through the opening on the coupling clamps. The locking lever is then screwed onto the screw thread. The height of the handles should be adjusted to suit the user. The height can be changed by pushing in and pulling out the handlebars. With the user standing erect the handles should be at the level of the wrists. Make sure that the handles are always aligned parallel to the direction of movement (pic. 1). Pull the locking lever clockwise to tighten it up. In doing so make sure that the groove on the head of the screw fits into the slot on the coupling clamp (pic 2).
5. Now adjust the levers so that the ends point downwards by releasing and turning them (pic. 5). This does not alter the securing effect of the levers!
6. The basket is hung on the hooks at the right and left below the seat (pic. 8).
7. Insert the tray with the little »feet« underside in the holes of the seat (pic. 8).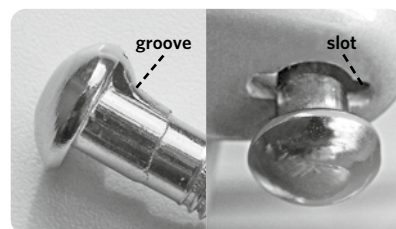


pic. 2


Use

Unfold/fold

- When opening out and when folding the rollator, make sure that no clothing or parts of the body become trapped. The rollator can be un-folded by pressing one of the brackets of the back wheels (pic. 3, point 1). Then push the crossbar down (pic. 3, point 2). The fold lock (pic. 3, point 3) automatically clicks into place if the cross strut is in the correct position (pic. 4).
- Check that the fold lock is firmly in position in order to prevent the rollator from folding together in an unwanted manner. When folding the rollator remove the shopping basket and the tray first. Press the fold lock up (pic. 4, point 1) and pull the crossbar up straight (pic. 4, point 2).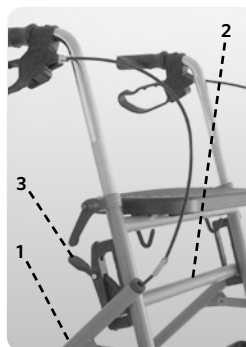

pic. 3

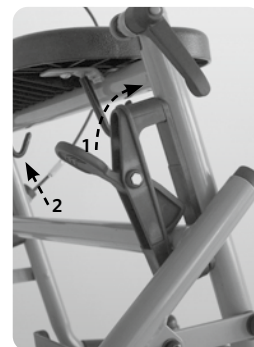pic. 4

Security lever

- The security levers allow the height of the handles to be adjusted individually. When turned anticlockwise they loosen and the handles can be pushed further in or pulled out (pic. 5, point 1).
- Turn the security levers clockwise and they will tighten (pic. 5, point 2).
- Now adjust the levers so that the ends point downwards by releasing and turning them. This does not alter the securing effect of the levers!


pic. 5

Brake (pic. 6)

The built-in brake functions as an operating brake for slowing down when moving. The parking brake can be used when the rollator is stopped.

1. Speed is reduced by pulling the brake lever upwards (operating brake) (pic 6, point 2).
2. The parking brake is activated by pressing the brake lever down until it engages (pic 6, point 1).
3. The parking brake is released from its locked position by pulling the brake lever upwards.
4. The rollator should not be pushed while the operating or parking brakes have been applied. If the rollator is pushed while the operating or parking brakes have been applied, the tyres and brakes may be subject to excessive wear and deteriorate.
5. Regularly check that the brakes work satisfactorily. We recommend checking before every use. The rollator must not be used with damaged brakes or damaged wheels.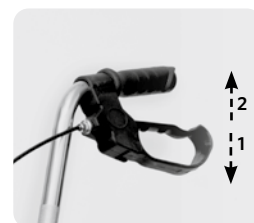


pic. 6

Brake adjustment (pic. 7)

If the braking force is decreasing due to wear on the wheels, the brake shoe can be adjusted. The brakes must be adjusted so that both wheels brake evenly. The distance between the brake pad and the wheel surface should be about 2 mm. After replacing wheels, the brake setting must be checked again and if necessary, readjusted.

1. Loosen the locking nut.
2. Adjust the braking force by turning the brake cable spanner inwards or outwards.
3. Check the braking function by turning the wheels and operating the brake levers. The wheels should turn freely but stop immediately when the brakes are applied. The wheels must not be able to move when the brakes are applied.
4. Tighten the locking nut firmly again.
5. If you are unsure about the correct adjustment of the brakes, please contact your specialist retailer.

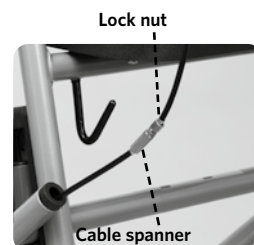pic. 7

Tray

The tray is designed only for small objects. The little »feet« under the tray enable it to be inserted securely in the holes of the seat. Do not place hot, heavy or sharp objects on the tray!

Warning: Do not use the tray as a seat!

Shopping basket

The basket allows the user to transport his shopping conveniently. The basket is hung on the hooks at the right and left below the seat.

Warning: Only use the basket to transport objects and do not overload it (max. 10 kg)!

In the right seating position (pic. 9) the seat enables the user to take short rests, thus providing

pic. 8

Seat

In the right seating position (pic. 9) the seat enables the user to take short rests, thus providing relief to tired legs and feet.

Warning: The seat is not a replacement for a chair! Place your weight on it only vertically and do not lean against it slantwise! Make sure that the brakes left and right are in 'parking'-position when you sit on the seat!

Do not lean against the seat! Do not lean over the sides or backwards while sitting on the rollator. That could impair the stability of the Rollator. Only sit on the seat with your back to the basket. Do not move the rollator when someone is sitting on the seat.

The seat has been tested for a user weight of up to 130 kg.

pic. 9

Overcoming obstacles with the support of the step aid

The step aid is intended for easier negotiation of flat steps and curbs. It is already mounted on the right side of the frame next to the rear wheel. Before using the step aid, make sure that you can stand safely on one leg. Practice using it with an assistant if necessary.

If you have to overcome an obstacle, for example a curb, drive the rollator straight towards the obstacle until the two front wheels are parallel directly in front of the obstacle. Pull the brake levers up with both hands so that both rear wheels are braked. Now use your right foot to carefully push down the step assistance and simultaneously pull both handles toward you. When the rollator's center of gravity is above the rear wheels, release the brake levers simultaneously and slowly so that you can let the rollator roll on its rear wheels toward the obstacle. Apply the brakes evenly again. Now you can lower the front wheels onto the obstacle with the brakes applied. Release the brakes and push the rollator forward until it is completely on the obstacle or the obstacle is completely overcome.


pic. 10

General care and cleaning

Rollator

The rollator can be cleaned with lukewarm water, neutral commercial and household cleaning agents and a soft cloth to remove dirt, then rinsed and dried. No water must get into the pipes. After cleaning or after rain, dry the walker thoroughly with a dry, clean cloth. Also clean the brake pads and springs regularly. Do not use abrasive, corrosive or bleaching cleaning agents. Do not use metal brushes or other sharp-edged cleaning aids for cleaning. Further cleaning instructions for reuse and before changing patients can be found under the Hygiene Plan.

Do not use the rollator in temperatures below -35°C or above $+60^{\circ}\text{C}$.

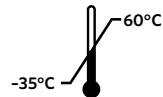Re-use

The product can be re-used. It must be cleaned and disinfected according to instructions. It must be inspected by specialist shops for any faults and then authorised before it can be used again. Please order our hygiene and maintenance guidelines. Send your rollator for regular checks to your dealer. He will be able to detect any wear and tear and replace parts, if necessary. If you find the brake action to be faulty in any way, please contact your dealer immediately.

Please find below a maintenance table and hygiene plan for the regular care and maintenance of this device:

Maintenance Table	
When	
Before every re-use	
What	Caution
Check general condition Visually inspect all frame parts including welded joints for deformation, damage, wear or corrosion	If you find any defects, replace the affected parts of the device.
Check hand grips visually and by attempting to turn them	If you find that the hand grips are damaged or no longer firmly connected to the frame, they must be replaced.
Check the tyre tread and ball bearings	If you find that the wheels are worn or cracked, they must be replaced.
Check the folding lock	If the folding mechanism no longer works properly, it must be repaired or replaced.
Check the connecting elements for tightness	If you find that screws, nuts, rivets, etc. are loose or missing they must be tightened, fitted or replaced.
Check the braking system is working satisfactorily Apply the operating brake until it reaches the stopping point. The wheels must not be able to turn now when pushed hard. Apply the parking brake by pressing the brake levers down until it engages. The wheels must not be able to turn now when pushed hard.	If the brakes are defective, the brake slack, brake cables, brake shoes and wheels must be checked and the defective part of the device must be replaced if necessary.

Hygiene Plan

The rollator must be cleaned and disinfected before each re-use or before each change of patient.

The disinfection of the rollator must only be carried out by a specialist.

Cleaning/disinfection

1. Put on protective gloves and, if necessary, protective clothing (disposable apron, mouth and nose protector, etc.) before preparing the rollator.
2. Always clean first, then disinfect.
3. To remove any dirt, the rollator can be cleaned with lukewarm water, neutral, commercial and household cleaning agents and a soft cloth, then rinsed and dried. No water must get into the pipes. Also clean the brake pads and the springs. Do not use abrasive, corrosive or bleaching cleaning agents. Do not use metal brushes or other sharp-edged cleaning aids for cleaning. After using cleaning agents, rinse the rollator thoroughly with clean water. Before disinfecting, there must be no residues of cleaning agents on the surface. After cleaning, dry the rollator with a dry, clean cloth.
4. Wipe or spray the rollator with a disinfectant listed by DGHM/VAH (follow the manufacturer's instructions).
Disinfectants containing chlorine or phenol as well as vinegar-based cleaning agents must not be used. Wipe disinfection is always preferable to spray disinfection. After the disinfectant has taken effect, dry the rollator with a dry, clean cloth.

Storage

Store the Rollator light and dust-protected in a dry place. Do not store near a heat source. Protect the rollator from dirt and contamination after treatment or if it will not be used for a long period of time. ☂️ ☀️

Disposal

Do not dispose of the rollator in the normal household rubbish. Ask your local recycling authority how to dispose of it properly and in an environmentally friendly way.

Technical Details

Material:

- Frame: Steel
- Wheels: Thermoplastic rubber (TPR)
- Dimensons: 58,3 x 68,5 x 79,5–95,6 cm (W x L x H)
- Folding measure: 26,5 x 58,3 x 90 cm (W x L x H)
- Wheels: 19,2 x 3 cm
- Seat level: 61,5 cm
- Distance between handles: 41 cm
- Weight: 8,92 kg without basket and tray, 9,88 kg incl. basket and tray
- Turning cycle: 82 cm
-  130 kg

Basket

- Material: coated wire
- Dimensions: 40 x 30 x 17,5 cm (W x L x H)
-  10 kg

Tray

- Material: Polypropylene (PP)
- Dimensions: 25,6 x 39 x 3,7 cm (W x L x H)
-  5 kg

Product marking

The following table explains the symbols used for labelling RUSSKA products. The translation key refers to our entire product range - please refer to the labelling of the product you have purchased to find out which symbols are actually relevant.

Symbol	Meaning	Symbol	Meaning
	Manufacturer of the product		Unique product identification (Unique Device Identification)
	Authorised representative in Switzerland		Maximum user weight
	Importer		Maximum luggage load
	Sales partner		Maximum width
	CE marking in accordance with directive 2017/745/EU on medical devices		Designed for indoor and outdoor use
	Medical device		Maximum load capacity
	Observe instructions for use		Maximum gripping load/load capacity
	Serial number		Maximum stress force
	Batch designation		Natural rubber latex is included
	Catalogue number		Protect from sunlight
	Date of manufacture		Store in a dry place

 This device complies with Regulation (EU) 2017/745 on medical devices and bears a CE marking. 

If a serious incident occurs involving this medical device, contact the manufacturer of this device at info@russka.de as well as the national competent authority in your country.

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