entire lower limb. If wrinkles should form, distribute the material as evenly as possible over all areas of your leg, smoothing it upwards from your ankle.

G If you are wearing compression tights, pull the second leg of the tights on in the same way as the first. Once both sides have been pulled up to below the knee, turn the tights back to the right side and pull the individual sections upwards evenly on both legs. Then adjust the fit of the knitted fabric with the flat of your hand.

H Now pull the bodice up to the waist. The gusset should be vertical in the crotch. Important: The tights must not be overstretched. Finally, spread the knitted fabric evenly from bottom to top.

Intended purpose
Compression therapy involves measures to apply extensive, external pressure to treat vein problems and lymph drainage disorders. The aim of the extensive pressure is to prevent the formation of oedema and to support venous return and lymph drainage. Compression garments exert an optimal pressure gradient that gradually decreases from bottom to top. Compression therapy is primarily used to treat vein problems and oedema. It involves the use of compressive and elastic double-stretch fabrics or knitted fabrics that wrap right round the extremities. As a general rule, compression therapy is a long-term treatment requiring appropriate products that are used by the patients themselves when medically indicated and on the instructions from a doctor.

Specific instructions for putting on stockings with gripper elastic
In order to put the gripper elastic on more easily, turn it over twice and do not grip the seam directly

when putting on or taking off. Please correct the position of the gripper elastic when turning down or rolling up. The gripper elastic fits perfectly on non-creamed, non-oiled and slightly hairy skin.

Instructions for taking off
Grip the stocking at the top edge from the inside and outside, simultaneously on the left and right-hand sides. Now pull the stocking downwards to the heel. When doing this, the inside of the stocking will move to the outside. Then, grip the area to the left and right of the Achilles heel, pull the stocking over the heel and then take the stocking off completely.

Important
To prevent any loose threads and ladders caused by the fingernails, we recommend that you wear textured rubber gloves that are obtainable from your healthcare supply store. Knots and thread ends in the stocking are manufacturer elated and must not be cut off or pulled. Rings, sharp toenails and fingernails and defective shoes must be avoided. Keep away from fire. **Please wash before first use.** The product may only be used on intact skin, wounds must be covered if necessary. The medical compression stocking must not be used in wet areas. Sale and adjustment of the product may only be carried out by trained specialists. Combination with other products is possible. This should be discussed with the attending physician or specialist. Wearing the medical compression stocking is not suitable for sports activities or for very active patients. To get used to the product, it is recommended to wear it for hours at an initial time.

Indications
• Chronic venous diseases: Improvement of venous symptoms, improvement of

quality of life in chronic venous diseases, prevention and therapy of venous oedema, prevention and therapy of venous skin changes, eczema and pigmentation, dermatoliposclerosis and atrophie blanche, therapy for venous leg ulcer, therapy for mixed (arterial and venous) leg ulcers (taking into account contraindications), prevention of venous leg ulcer recurrence, pain reduction in venous leg ulcers, varicosis, initial phase after varicose therapy, functional venous insufficiency (in obesity, sedentary occupations, standing occupations), venous malformations

- Thromboembolic venous diseases: Superficial venous thrombosis, deep vein thrombosis, condition after thrombosis, post-thrombotic syndrome, thrombosis prophylaxis in mobile patients
- Oedemas: Lymphoedema, oedema during pregnancy, post-traumatic oedema, post-operative oedema, post-operative reperfusion oedema, cyclic idiopathic oedema, lipoedema, congestion due to immobility (arthrogenic congestion syndrome, paresis and partial paresis of the limb), occupational oedema (standing occupations, sedentary occupations), oedema due to medication if no change is possible
- Other indications: Obesity with functional venous insufficiency, inflammatory dermatoses of the legs, nausea, dizziness in pregnancy, congestive disorders in pregnancy
- In case of pregnancy, changing abdomen circumferences mean that a pregnancy body part should be exclusively used for treatment with pantyhose.

Contraindications
Absolute Contraindications:
Advanced peripheral (arterial) occlusive disease, decompensated cardiac insufficiency, optic phlebitis, phlegmasia coerulea dolens.

Relative Contraindications: Weeping dermatoses, intolerance of compression stocking material, sensory disturbances of the extremity, advanced peripheral neuropathy (e. g. diabetes mellitus), primary chronic polyarthritis.

Life cycle/ Use period
The life cycle of the product is determined by the natural wear and tear that occurs with correct use. The life cycle is usually 6 months.

Transport and storage
Store and transport in a dry place that is protected against light and damage. A 3-year shelf life is guaranteed under correct storage conditions.

Warranty
The legal regulations of the country in which you purchased the product apply to this product. If you suspect a warranty case, please contact your specialist dealer. Warranty is excluded in case of use not corresponding to the indication, failure to observe the risks associated with use or unauthorised modifications to the product.

Side effects
There are no known side effects when using this product correctly. In addition, the medical compression stocking should be regularly checked for damage or wear by the user. This aid is only intended for the treatment of one patient. To ensure that the treatment is successful, regular check-ups by your doctor are required. Very rarely, medical compression stockings can cause allergic reactions. If acute leg pain or skin irritations should occur whilst the stockings are being worn, please consult your doctor immediately.

Obligation to report
If a serious deterioration in the health condition occurs during use

of this medical device, we kindly ask you to report this to the specialist dealer or to us directly as the manufacturer as well as to the BfArM (Federal Institute for Drugs and Medical Devices). You can find our contact details at the end of these instructions for use.

Disposal
This product must be disposed of in accordance with local regulations at the end of its use.

Washing instructions
Please wash the compression stockings before wearing them for the first time. To wash them, turn

them inside out and put them in a laundry bag. For gentle care, we recommend daily hand washing or delicates washing with preferably Schiebler special detergent; fabric softener or bleach must not be used. Please rinse the stockings thoroughly, press them out between two towels and hang them up loosely. Please do not wring, do not dry on the heater or in the sun, as this will affect the durability. The compression stockings must not be dry cleaned. Please follow the care instructions on the sewn-in textile label. The exact material composition can be found on the sewn-in textile label.

Gültig für / Valid for: Veni KKL 1+2, Venex Micro Light KKL 1+2, Venesso KKL 1+2, Doktus KKL 2+3, Star Cotton KKL 1+2, Veni Dur Extra KKL 3, Business & Holiday KKL 2, Pertex Leicht KKL 1, Doktus mit Naht / DN1 KKL 2, Goldpunkt Karat / GP 75 KKL 2, Goldpunkt KKL 3 + 4

Heinz Schiebler GmbH & Co. KG
Marienallee 74 | D-24937 Flensburg
Tel. +49 (0)461 978 99 90

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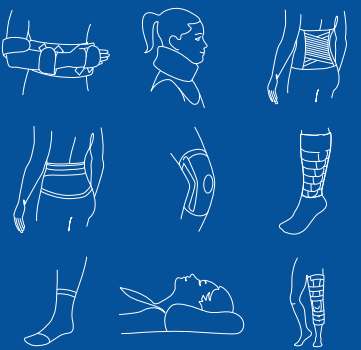
Our product, your choice. Thanks a lot.

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EN

Donning instructions for stockings with open toe

➊ Slip your foot into the donning aid provided with the stockings. Hold the heel and turn the stocking inside out as far as the toe.

➋ Using both hands, place the stocking over your foot, ensuring

that the heel sits correctly. Straighten the stocking carefully and place it fully over your leg, please take care not to pull the stocking too roughly.

➌ Remove the donning aid through the open toe.

Please also see our training video „Proper application of medical compression stockings“ on our website www.schiebler.de

Donning instructions for stockings with closed toe

➊ To turn the stocking inside out, carefully reach into the stocking right down to the heel.

➋ Hold the heel firmly and turn the stocking inside out as far as the toe.

➌ Put the toe of the stocking over your toes. Stretch the foot of the stocking and gently pull it over your foot.

➍ Make sure that the heel of the stocking sits correctly. Using both hands, slowly pull the rest of the stocking up along your shin.

➎ Distribute the fabric evenly along your leg.

➏ Do not pull the stocking up tight, distribute it evenly over the